

AVANT QUICK KEYS

ACTION	KEY	COMMENTS
Help	F1	Gives context help
Exit program	Alt-F4	Exits the program. Will prompt to save data
Go back to previous screen	Ctrl-Back	Same as clicking Back button
Discard a curve	Del	Same as clicking on Discard button. Select desired curve with the mouse first
Discard all curves	Ctrl-Del	Same as holding Ctrl key and clicking on the Discard icon
Discard all curves from all ears	Ctrl-Shift-Del	For dual-plot screens, discard all curves from both plots
Scroll plot	Ctrl-Up / Ctrl-Down or Mouse Wheel	Only on plots with scrolling available.
Toggle Ears	F2	
Change ear to Left	Ctrl-L	
Change ear to Right	Ctrl-R	
Copy to Clipboard	Ctrl-C or Ctrl-Insert	Copy audiometry data to Clipboard. Valid only on Audiometry screens.
Paste from Clipboard	Ctrl-V or Shift-Insert	Paste audiometry data from Clipboard. Valid only on Audiometry screens.
Switch Single/Dual Plots	Ctrl-1 for single, Ctrl-2 for dual, Ctrl-~ to toggle	Works only on screens that support dual plot mode.

VIDEO QUICK KEYS

ACTION	KEY	COMMENTS
Video preview	Shift-Enter	Same as clicking on the camera button on the toolbar
Capture Picture	F2	Only for live video screens
Run/Freeze	F3	Only for live video screens
Capture Picture	Any key when running in Single Click mode	Only for video screens when running in Single Click capture mode

AUDIOMETRY QUICK KEYS

ACTION	KEY	COMMENTS
Start signal	Space or Enter or S	Same as clicking on the Start button
Start signal with the mouse	Point the cursor at the desired frequency/level points, push and hold Left mouse button, and then push the Right mouse button	
Increase level	ArrowUp or W or + (plus)	In Tone Audiometry, holding the Up or Down arrows will start a signal if AutoSound is selected in the Audiometry Options. Arrow keys can be reversed in the Audiometry options. Hold the Shift key to move levels by the Level Unit set in options.
Decrease level	Arrow Down or Z or - (minus)	
Increase masking level	Ctrl-Arrow Up or Ctrl-W	For dual-channel tests (Stenger, WR) controls Right channel. Hold Shift key to move levels by the Level Unit set in the options.
Decrease masking level	Ctrl-Arrow Down or Ctrl-Z	
Move frequency left	Left Arrow or A	
Move frequency right	Right Arrow or D	
Masking on/off	M	
Toggle channel lock	K	
No response	N	For tone audiometry only
Could Not Test	Ctrl-N	For tone audiometry only
Did Not Test	Ctrl-Shift-N	For tone audiometry only
Talk Forward	T or F12	
Monitors	O	
Left ear	L or Ctrl-L	
Right ear	R or Ctrl-R	
Toggle ears	G or F2	
Switch Signal Type	Shift-F1..F3	Only where Tone, Warble, NBN is available
Score Tone Point	Insert	Can be used if Auto-Record Tone Points is turned off

SPEECH AUDIOMETRY

ACTION	KEY	COMMENTS
Increase level	Arrow Right or Arrow Up or W or + (plus)	
Decrease level	Arrow Left or Arrow Down or Z or - (minus)	
Increase masking level	Ctrl-Up or Ctrl-W	
Decrease masking level	Ctrl-Down or Ctrl-Z	
Toggle channel lock	K	
Correct answer	F7	
Incorrect answer	F8	
Play word	F9	For Word Lists
Play next word	F10	For Word Lists
Talk Forward	T or F12	
Monitors	O	
Left ear	L or Ctrl-L	
Right ear	R or Ctrl-R	
Toggle ears	G	

In both Tone and Speech Audiometry, hold the Ctrl key and click with Left mouse button to manually add a point, or Right button to remove a point.

Any Underlined letter in the drop down menus of the toolbar is the hot key for that option. Press and hold down the Alt key and press the underline letter to choose the function.