

## STUDIO SOFTWARE QUICK KEYS

### GENERAL

| ACTION                     | KEY  | COMMENTS   |
|----------------------------|--|--|
| Help                       | F1   | Gives context help                                 |
| Exit program               | Alt-F4   | Exits the program. Will prompt to save data        |
| Go back to previous screen | Ctrl-Back  | Same as clicking Back button                       |
| Scroll plot                | Ctrl-Up / Ctrl-Down or Mouse Wheel                   | Only on plots with scrolling available             |
| Toggle Ears                | F2   |  |
| Change ear to Left         | Ctrl-L   |  |
| Change ear to Right        | Ctrl-R   |  |
| Switch Single/Dual Plots   | Ctrl-1 for single, Ctrl-2 for dual, Ctrl-~ to toggle | Works only on screens that support dual plot mode. |
| Print                      | Ctrl-P   | Print with template selection                      |
| Quick Print                | Ctrl-Shift-P   | Print with default template                        |

### VIDEO QUICK KEYS

| ACTION                     | KEY                         | COMMENTS   |
|----------------------------|-----------------------------|--|
| Capture Picture            | Enter or Space or F2 or F12 | Only for live video. Captures for currently selected side                              |
| Capture Left Side Picture  | F8                          | Captures Left Side picture   |
| Capture Right Side Picture | F9                          | Captures Right Side picture  |
| Run/Freeze                 | F3 or F11                   | Only for live video  |
| Save as Picture File       | Ctrl-S                      | Save active picture as file. JPEG, BMP, PNG, GIF, and some other formats are supported |
| Open Picture File          | Ctrl-O                      | Open a picture file  |
| Copy to Clipboard          | Ctrl-C                      | Copy active image to Windows Clipboard   |
| Paste from Clipboard       | Ctrl-V                      | Paste picture from Windows Clipboard   |

## AUDIOMETRY QUICK KEYS

| ACTION                         | KEY   | COMMENTS   |
|--------------------------------|---|--|
| Start signal                   | Space or Enter or S   | Same as clicking on the Start button   |
| Stop signal                    | ESC   |  |
| Start signal with the mouse    | Point the cursor at the desired frequency/level points, push and hold Left mouse button, and then push the Right mouse button |  |
| Increase level                 | ArrowUp or W or + (plus)  | In Tone Audiometry, holding the Up or Down arrows will start a signal if AutoSound is selected in the Audiometry Options. Arrow keys can be reversed in the Audiometry options. Hold the Shift key to move levels by the Level Unit (set in options) |
| Decrease level                 | Arrow Down or Z or - (minus)  |  |
| Increase masking level         | Ctrl-Arrow Up or Ctrl-W or Numpad2 or Ctrl + Numpad + (plus)  | For dual-channel tests (Stenger, WR) controls Right channel. Hold Shift key to move levels by the Level Unit (set in the options)  |
| Decrease masking level         | Ctrl-Arrow Down or Ctrl-Z or Num pad8 or Ctrl + [Numpad - (minus)]  |  |
| Move frequency left            | Left Arrow or A   |  |
| Move frequency right           | Right Arrow or D  |  |
| Masking on/off                 | M   |  |
| Toggle channel lock            | K   |  |
| No response                    | N   | For tone audiometry only   |
| Talk Forward                   | T or F12  |  |
| Monitors                       | O   |  |
| Toggle Phonemes View           | P   |  |
| Toggle Hearing Loss Level View | I   |  |
| Toggle Banana Curve View       | B   |  |
| Left ear                       | L   |  |
| Right ear                      | R   |  |
| Toggle ears                    | G   |  |

|                    |                             |  |
|--------------------|-----------------------------|--|
| Switch Signal Type | Shift-F1..F3                | Only where Tone, Warble, NBN is available            |
| Score Tone Point   | Insert or H                 | Can be used if Auto-Record Tone Points is turned off |
| Delete Tone Point  | Delete or Mouse Right Click |  |
| Delete Curves      | Ctrl + Delete               |  |

## SPEECH AUDIOMETRY

| ACTION                 | KEY  | COMMENTS       |
|------------------------|--|----------------|
| Increase level         | Arrow Right or Arrow Up or W or + (plus)   |                |
| Decrease level         | Arrow Left or Arrow Down or Z or - (minus) |                |
| Increase masking level | Ctrl-Up or Ctrl-W                          |                |
| Decrease masking level | Ctrl-Down or Ctrl-Z                        |                |
| Toggle channel lock    | K  |                |
| Correct answer         | F7   |                |
| Incorrect answer       | F8   |                |
| Play word              | F9   | For Word Lists |
| Play next word         | F10  | For Word Lists |
| Talk Forward           | T or F12                                   |                |
| Monitors               | O  |                |
| Left ear               | L  |                |
| Right ear              | R  |                |
| Toggle ears            | G  |                |
|                        |  |                |

## QUICK SIN

| ACTION                | KEY | COMMENTS |
|-----------------------|-----|----------|
| Check QuickSin Word 1 | 1   |          |
| Check QuickSin Word 2 | 2   |          |
| Check QuickSin Word 3 | 3   |          |
| Check QuickSin Word 4 | 4   |          |
| Check QuickSin Word 5 | 5   |          |
|                       |     |          |