









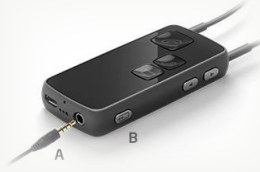


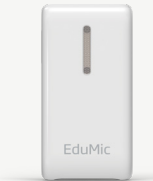



Oticon hearing aid and connectivity overview

Hearing aids	Accessory options	Hardwire options	Notes
 <p>Sensei Family</p>	 <p>Streamer Pro</p>	 <p>AP900 adapter & Audio cord</p>	<ul style="list-style-type: none"> • The audiologist must link the Streamer Pro to the child's hearing aid(s) in our software. • The AP900 adapter slides on to the bottom of the hearing aid. View video here: https://youtu.be/-0sVvVzJcM • The audio cord plugs into the AP900. The prongs only fit one way.
 <p>Opn Play™, Opn S™, Xceed, Xceed Play, Ruby, Opn™ and Siya BTE Styles</p>	 <p>ConnectClip or EduMic (paired wirelessly to the hearing aids)</p>	 <p>AP1000 adapter & Audio cord</p>	<ul style="list-style-type: none"> • ConnectClip and EduMic can be paired to hearing aids at home. This must be done before pairing the accessory to other technology. • The AP1000 adapter requires a specific battery door on the hearing aid. Contact your audiologist to ensure you have correct door. View video here: https://youtu.be/gNAs55H9CDw • To pair ConnectClip to hearing aids, see: https://youtu.be/38ScqRRXX6c • To pair EduMic to hearing aids, see: https://youtu.be/2xRzR19P18o
 <p>Oticon Play PX, Oticon More™, and Oticon Zircon miniBTE and miniRITE Styles, Opn Play, Opn S, Ruby, Opn and Siya miniRITE Styles</p>	 <p>ConnectClip or EduMic (paired wirelessly to the hearing aids)</p>	<p>NONE AVAILABLE</p>	<ul style="list-style-type: none"> • To pair EduMic to hearing aids, see: https://youtu.be/2xRzR19P18o • To pair ConnectClip to hearing aids, see: https://youtu.be/38ScqRRXX6c

Please call the Consumer Hotline at 855.400.9766 if you need assistance with connectivity.

Oticon accessory and connectivity overview

Accessory	Bluetooth® (BT) connectivity	Hardwire connectivity	Notes
 <p>Streamer Pro</p>	 <p>The Streamer Pro can pair to a phone, tablet or computer with existing BT capability. https://youtu.be/Dxa89bjrS4o</p>	 <p>Streamer Pro can hardwire into anything with a standard 3.5mm audio port (cable included in box). https://youtu.be/_7aE5SpU0Is</p>	<ul style="list-style-type: none"> • Streamer Pro must be worn around the child's neck while in use. • For additional Streamer Pro instructions and videos, please visit: www.oticon.com/solutions/for-children/connectline-children
 <p>ConnectClip</p>	 <p>The ConnectClip can pair to a phone and some computers and tablets with existing Bluetooth capability. https://youtu.be/cGSYnpzjNws</p>	<p>NONE AVAILABLE</p>	<ul style="list-style-type: none"> • Devices with limited or no Bluetooth require a USB port and a Sennheiser BTD 800 dongle (can be purchased through your audiologist or online through various retailers). • How to pair ConnectClip to BTD 800 dongle: https://youtu.be/sAUwVBHuKVg • For additional ConnectClip instructions and videos, please visit: https://www.oticon.com/support/how-to/use-connectclip
 <p>EduMic</p>	<p>NONE AVAILABLE (Only connects to Oticon Bluetooth hearing aids on the Velox™, Velox S™ and Polaris™ platforms)</p>	 <p>EduMic can hardwire into anything with a standard 3.5mm audio port (cable included in box). https://youtu.be/hj536ySID8A</p>	<ul style="list-style-type: none"> • For additional EduMic instructions and videos, please visit: https://www.oticon.com/support/how-to/use-edumic

Please call the Consumer Hotline at 855.400.9766 if you need assistance with connectivity.