

# Oticon | Streamer

## INSTRUCTIONS FOR USE

## STREAMER 1.4

ADDENDUM TO INSTRUCTIONS FOR USE  
STREAMER 1.4



 **Bluetooth**<sup>®</sup>



**oticon**  
PEOPLE FIRST

## Contents

<u>Transfer call to cell phone</u>	<u>4</u>
<u>Start wireless audio from Streamer</u>	<u>5</u>
<u>Resume ConnectLine Microphone from Streamer</u>	<u>6</u>
<u>Connect/Bluetooth indicator always fading</u>	<u>7</u>


## Transfer call to cell phone

Most cell phones support the transfer call function.

<b>Long press</b>	When in a call a long press on the phone button will transfer the call to the cell phone. This is indicated by the phone button blinking. A subsequent long press will transfer the call back to the Streamer.  When transferring a call to the cell phone this typically has to be acknowledged/accepted on the cell phone.	
<b>Short press</b>	Short press will hang up the call even though transferred back to the cell phone.	

## Start Wireless audio from Streamer


If Streamer has been paired to an A2DP Source (e.g., MP3, PC, or other streamed audio).

<b>Long press (&gt; 0.5 sec)</b>	Will start audio streaming from paired audio source (A2DP).	
<b>Short press (&lt; 0.5 sec)</b>	Will stop audio streaming from active source.	

For additional information about audio streaming - please refer to section 6.4 of the Instructions For Use - Streamer 1.2.

## Resume ConnectLine Microphone from Streamer





If Streamer has been paired to a ConnectLine Microphone and this is in range and power is on.

<b>Double press</b>	Will start/resume sound from ConnectLine Microphone.	
<b>Short press</b>	Will stop audio streaming from ConnectLine Microphone.	

For additional information about ConnectLine Microphone - please refer to the Instructions For Use - ConnectLine Microphone.

## Connect/Bluetooth indicator always fading

If Connect is on, then the Connect indicator will continue to fade when no key lock (the green indicator is exposed).

<b>No key lock</b> 	Connect indicator keeps fading.	
<b>Key lock</b> 	Connect indicator off. Phone and audio button dimmed to 25% intensity.	

## People First

People First is our promise  
to empower people  
to communicate freely,  
interact naturally and  
participate actively



0000110346000001



FSC

MIX  
Paper

FSC® C043970

**oticon**  
PEOPLE FIRST