


Native Controls for iOS and Oticon Hearing Aids

iPad® and iPhone® Devices

Oticon hearing aids with 2.4 GHz Bluetooth® technology can be used with Apple® devices to make adjustments to hearing aids including adjustments to volume, program changes and turn on Live Listen® (allowing you to turn your phone into a remote microphone).

To access these controls you can select **Settings > Accessibility > Hearing Devices** and then **select your hearing aids**. You can also access the controls by using the Accessibility Shortcut.

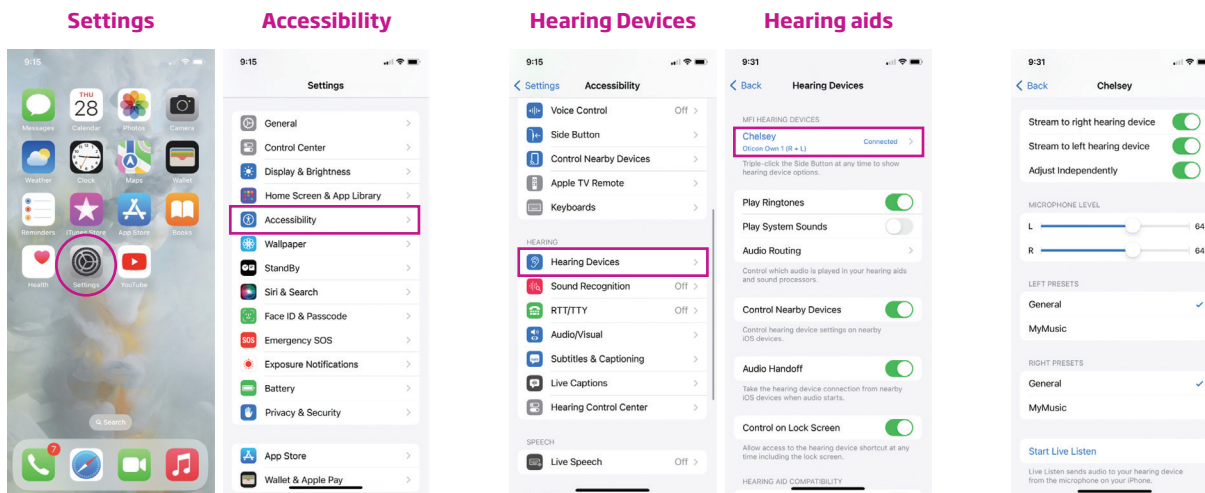

To set up the Accessibility Shortcut go to **Settings > Accessibility > Accessibility Shortcut** and from the list of items **select MFi Hearing Devices**. Once setup you will be able to access the hearing aid controls by triple clicking on the side button for iPhone X or later. For older devices quickly click on the Home button three times.

