


Instructions for use

Oticon Companion

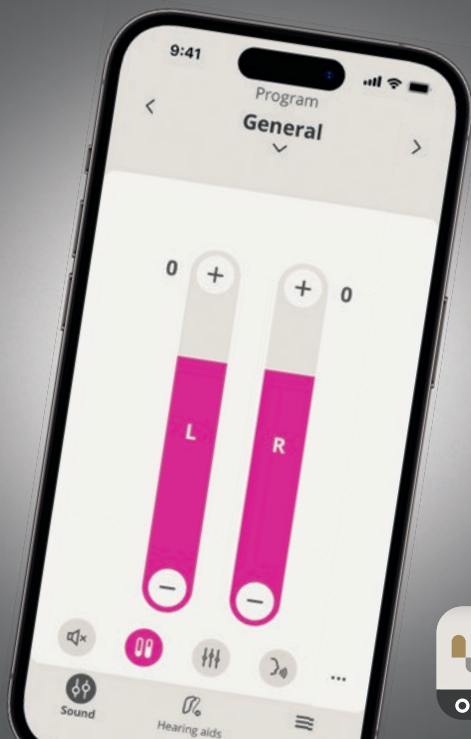

oticon
life-changing technology

Table of contents

About

Introduction	3
How to use this guide	5
Intended use	5
Disclaimer for Oticon Companion	6
System requirements	8

Daily Use

Change volume and mute	10
Change program	11
Charging status	12
HearingFitness™	13
Accessories and your app	15
Sound equalizer	17
Streaming equalizer	18
Remote microphone	19
SpeechBooster*	20
Hearing aid settings	21
Find my hearing aids	22
Apple Watch®	23
Create account or sign in	26

Remote Visit

Prepare for your remote visit	28
Begin visit	30
Waiting Room	30
Your RemoteCare visit	31
Hearing evaluation (In-situ audiometry)	32

Tinnitus

Intended use of Tinnitus SoundSupport	34
---------------------------------------	----

Warnings

General warnings	36
------------------	----

More info

Troubleshooting	37
Troubleshooting RemoteCare	38
Technical information	40

Introduction

The following Instructions for use are valid for Oticon Companion 1.5.0 or later.

Oticon Companion is designed for easy control of your hearing aids sound settings and allows you to control the sound streamed to them. The app also allows for remote communication between you and your hearing care professional.

The RemoteCare feature of this app allows you to have follow-up visits from the comfort of your home. This feature allows you to see and hear your hearing care professional, as well as send text messages during your remote visit. The RemoteCare feature also enables data communication between your hearing aids and your hearing care professional through a stable Internet connection. In certain cases, you may have to attend follow-up visits in person at your hearing care professional's practice.

Ensure you read this booklet carefully, including the **Warnings** section, before using the app. This will help you get the most benefit out of the app.



Warnings

Text marked with a warning symbol must be read before using the app.

When this guide mentions device(s), it refers to for example, an iPhone® or iPad®, or your Android™ device such as a phone or tablet.

The app runs on your iPhone, iPad or Android device. Ensure that you always update the app to the newest compatible version. Also, ensure that you update your device to the latest version of iOS or Android OS.

In case of any additional questions regarding the use of the app, contact your hearing care professional.

Be aware that some settings are dependent on the fitting done by the hearing care professional.

This guide helps you with:

- Getting started with the app
- Using the app on your device

| **About** | [Daily Use](#) | [Remote Visit](#) | [Tinnitus](#) | [Warnings](#) | [More info](#) |

How to use this guide

Be aware that you must download the app on your device *before* you follow the procedures in this guide.

To use most features of this app you will need to pair your hearing aids to your device first. If you need help pairing your device to your hearing aids visit: www.oticon.global/pairing

When using the RemoteCare feature, ensure that you follow the procedures in the **Remote Visit** section of this guide in chronological order. In most cases, the procedures are the same for all devices on both operating systems (iOS and Android).

Intended use

Intended use	Oticon Companion* is intended to give the user wireless access to the hearing aid through a mobile device for control of functionality and update of the hearing aid remotely. The product is intended to be used together with the given hearing solution.
Indications for use	There are no indications for use (diagnoses) for Oticon Companion itself.
Intended user	Persons with hearing loss using a wireless hearing aid and their caregivers. Children below 10 years of age are expected to be assisted by responsible caregivers.
Intended user group	Infants, children, and adults.
Use environment	Indoor or outdoor.
Contraindications	No contraindications.
Clinical benefits	See clinical benefits of the hearing aid.

* Oticon Companion is an accessory to a medical device (wireless hearing aid).

Disclaimer for Oticon Companion

The manufacturer does not take responsibility for the consequences of using this app outside its intended use or warnings.

Your hearing care professional is responsible for the fitting through the app.

The manufacturer does not take any responsibility for hearing aid fitting.

The manufacturer reserves the right to discontinue the app service without prior notice.

IMPORTANT NOTICES

The RemoteCare feature of this app connects you to your hearing care professional who sends updated settings to your hearing aids. If it is not possible to make all the changes remotely, then a physical visit to your hearing care professional is necessary.

The app should only be operated by the user of the hearing aid or responsible caregivers or hearing care professionals. Other third parties should be prevented from controlling the app.

The connection between hearing aids and cell phone may be lost due to signal interference, low battery, distance between phone and hearing aids etc. If this occurs and affects the expected functionality of the hearing aids, you are advised to reboot the hearing aids.

For details on the functionality of your specific hearing aid, refer to your hearing aid's Instructions for use.

The hearing care professional is responsible for obtaining the license needed to use the RemoteCare feature with the clients.

If you have additional questions about the use of the app, contact your hearing care professional.

System requirements

To use all the latest features of the app, you need the following:

- Compatible hearing aids with Bluetooth® Low Energy technology: See table on next page
- A compatible iPhone, iPad or Android device*
- For iPhone and iPad
 - iOS operating system software version 15.2 or later. To find out which version of iOS you have, on your iPhone or iPad, go to: **Settings > General > About > iOS Version**
- For Android devices
 - Android operating system software version 10.0 or later. To find out which version of Android you have, on your Android device, go to: **Settings > About phone > Software information**
- To use the RemoteCare feature: A stable Internet connection suitable for both sound and video streaming, with a recommended minimum speed of 1 Mbps (upload/download)

* To check for compatibility, visit: www.oticon.global/compatibility

Recommended firmware

Hearing instrument	Recommended firmware
Oticon Jet PX	1.0.0 +
Oticon Intent™	1.0.0 +
Oticon Real™	1.1.0 +
Oticon More™	1.4.0 +
Oticon Own™	1.4.0 +
Oticon Play PX	1.4.0 +
Oticon Zircon	1.4.0 +
Oticon Opn S™	7.0 +
Oticon Opn Play™	7.0 +
Oticon Xceed	8.0
Oticon Xceed Play	8.0
Oticon Ruby	1.0
Oticon Jet	1.0.0
Oticon Opn	6.1
Oticon Siya	1.1

+ = and later firmware versions

Features compatibility

- Charging status, Call handling (1)
- HearingFitness™, Sound equalizer, Remember volume settings (2)
- Streaming equalizer (3)
- Comfort pulse (4)

1 = Oticon Intent

2 = Oticon Intent, Oticon Real

3 = Oticon Jet PX, Oticon Intent, Real, More, Own, Play PX, Zircon, Oticon Opn S, Opn Play, Xceed, Xceed Play, Ruby

4 = Oticon Intent, Real, More, Own, Jet PX, Zircon


Change volume and mute


You can change the volume in both hearing aids at the same time or in each hearing aid independently.

Changing the volume only applies to the selected program.

To access volume and mute controls

- On the tab bar, tap **Sound**.

- Volume level indicator. The value **0** is the start-up volume as prescribed and set by your hearing care professional.
- To increase or decrease the volume, move the slider(s) up or down or press the +/- buttons.
- To switch between volume control in one or both hearing aids, tap the slider button .
- To mute or unmute the hearing aid microphones, tap the mute button.
- Tab bar
- Navigation bar
- Program selector

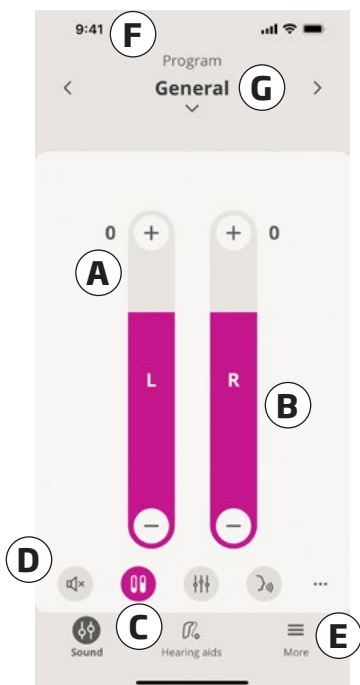

Change program

A program is a preset sound adjustment for your hearing aids. Each program is optimized for specific sound environments or remote sound sources.

Volume is adjusted independently for each program.

To change programs

1. On the tab bar, tap **Sound**.
2. On the navigation bar, in the program selector, tap the down arrow to see your available programs.
3. Tap the desired program to select it.

You can also toggle between your available programs either by tapping the right and left arrows in the navigation bar or by swiping to the left or right.

Your hearing care professional can define up to four different listening programs at your fitting session.

Additional programs for TV Adapter, EduMic and/or ConnectClip may be available. Tinnitus SoundSupport™ may also be available.

Charging status

You can monitor the charging status of your hearing aids in the app once they are placed in the charger*.

A persistent banner at the top of the app screen displays the charging state.

The banner turns green when your hearing aids are fully charged.

Some functionality is unavailable when charging:

- Hearing aid microphones are muted.
- Audio streaming is disabled.
- All settings under the **Sound** tab are disabled.

Note

* Some app features are only available for select hearing aids.

HearingFitness™

HearingFitness* is a feature that lets you set a daily goal for wearing your hearing aid. The feature will then measure your progress toward that goal.

For best results, ensure that you connect your hearing aids to the app each day.

First time use

1. On the tab bar, tap **More**.
2. On the **More benefits** screen, tap **HearingFitness**.
3. Tap **NEXT**, then tap **START HERE**.
4. On the **Set your daily goal** screen, near the top, tap a number to set your daily wear goal.
5. Tap **CONFIRM GOAL**.

The app is now setup to measure how long you wear your hearing aids each day.

Once you have set your wearing goal in HearingFitness, the app will automatically send you regular notifications on your progress.

You can disable these notifications or add additional notifications.

To disable or add additional notifications

1. On the tab bar, tap **More**.
2. On the **More benefits** screen, tap **Notifications**.
3. On the **Notifications** screen, scroll down to **HearingFitness**
4. Toggle your chosen notifications ON or OFF.

Note

* The HearingFitness feature is only available for select hearing aids. For further information, contact your hearing care professional.

HearingFitness

To see your current wearing goal progress

1. On the tab bar, tap **More**.
2. On the **More benefits** screen, tap **HearingFitness**.
3. The **Daily overview** screen now shows your progress toward your goal for the current day.

To see a daily, weekly, monthly or yearly view of wearing goal progress

1. On the tab bar, tap **More**.
2. On the **More benefits** screen, tap **HearingFitness**.
3. On the **Daily overview** screen, near the top, tap **Wearing time**.
4. On the wearing time screen, near the bottom, tap **CHANGE GOAL**.
5. On the **Wearing time** screen, near the top, tap the **D**, **W**, **M** or **Y** tab.

To adjust your daily wearing goal

1. On the tab bar, tap **More**.
2. On the **More benefits** screen, tap **HearingFitness**.
3. On the **Daily overview** screen, near the bottom, tap **CHANGE GOAL**.
4. On the **Set your daily goal** screen, near the top, tap a number to set your daily wear goal.
5. Tap **CONFIRM GOAL**.

Accessories and your app

If your hearing aids have been paired with one or more devices (TV Adapter, EduMic and/or ConnectClip), corresponding programs are available in the program selector.

TV Adapter/EduMic

If your hearing aids are paired with a TV Adapter you can use the app to control the sound streamed from your TV to your hearing aids.

If your hearing aids are paired with EduMic you can use the app to control the sound streamed from the device to your hearing aids.

ConnectClip

With your ConnectClip you can make hands-free phone calls and stream sound from any Bluetooth device to your hearing aids. Your ConnectClip also functions as a remote microphone.

To adjust the volume of sound streamed from your paired device


1. On the tab bar, tap **Sound**.
2. On the navigation bar, in the program selector, tap the down arrow, then tap **TV Adapter/EduMic** or **ConnectClip**.
3. On the navigation bar, tap the **Streaming** tab.
4. Move the slider(s) up or down.

To adjust the volume of ambient environmental sounds picked up by your hearing aids

1. On the navigation bar, tap the **Surroundings** tab.
2. Move the slider(s) up or down.

To switch between streaming sources

If you have more than one TV and a compatible device (TV Adapter/EduMic) paired to your hearing aids, you can use the app to switch between streaming sources.

1. On the tab bar, tap **Sound**.
2. On the navigation bar, in the program selector, tap the down arrow, then tap **TV Adapter/EduMic**.
3. On the bottom, above the tab bar, tap the **Devices** button . The highlighted device is the one currently streaming.
4. From the list, tap the desired streaming source.


To rename a connected device

1. On the bottom, above the tab bar, tap the **Devices** button.
2. Tap the pencil icon for the device you want to rename.
3. In the text box, enter the desired name.
4. Click **Save**.

Sound equalizer

Sound equalizer balances the pitch of the surrounding, external sounds you hear through your hearing aids.

To adjust the sound equalizer

1. On the tab bar, tap **Sound**.
2. Above the tab bar, in the middle, tap the sound equalizer button .
3. Move the slider(s) up or down.
 - **Low** equalizes bass tones.
 - **Mid** equalizes mid-range tones.
 - **High** equalizes treble tones.

Default settings are always 0 dB and the range is between +6 dB and -6 dB.

Streaming equalizer


Streaming equalizer lets you adjust the loudness of frequency bands for the sound streamed from your mobile devices, TV Adapter, EduMic and/or ConnectClip.

Streaming equalizer is not available on any Tinnitus program.

Note

The streaming equalizer button is always visible on iPhone, iPad and selected Android devices. On all other Android devices, the button appears the first time you stream sound from your phone or other connected devices.

To adjust the streaming equalizer

1. On the tab bar, tap **Sound**.
2. Above the tab bar, within the row of buttons, tap the streaming equalizer button .
3. Move the slider(s) up or down.
 - **Low** equalizes bass tones.
 - **Mid** equalizes mid-range tones.
 - **High** equalizes treble tones.

Default settings are always 0 dB and the range is between +6 dB and -12 dB.


Remote microphone

This functionality lets you use your iPhone or iPad as a remote microphone.

For example, you can place your iPhone near a person speaking across a room and the sound is streamed to your hearing aids.

Remote microphone is not available on Tinnitus, ConnectClip or TV Adapter/EduMic programs.

To activate or deactivate the remote microphone

1. On the tab bar, tap **Sound**
2. On the navigation bar, in the program selector, tap the down arrow and select one of the general programs.
3. On the bottom, above the tab bar, within the row of buttons, tap the microphone button , then toggle the switch.


Note

The remote microphone functionality is currently only available for iPhone and iPad.

SpeechBooster*

SpeechBooster helps you focus on speech in noisy environments. When activated, this feature enhances speech by suppressing non-speech sound sources from the sides and back.

To activate and deactivate SpeechBooster

- Above the tab bar, on the right, tap the SpeechBooster button , then toggle the switch.

(optional) Under the toggle switch, tick the box. You can then activate SpeechBooster by tapping the icon on the tab bar.

The feature is deactivated when you change programs, open the battery drawer/place the hearing aids in the charger, or reconnect the hearing aids.

Note

* The SpeechBooster feature is only available for select hearing aids. For further information, contact your hearing care professional.

Hearing aid settings

You can use the app to control a variety of settings in your hearing aids. You can set up battery status notifications and control how your hearing aids handle volume and call handling.

To start:

1. On the tab bar, tap **Hearing aids**.
2. On the **Battery level** screen, near the bottom, tap **Hearing aid settings**.

To activate or deactivate battery status notifications

1. On the **Hearing aid settings** screen, tap **Battery notification settings**.
2. On the **Battery status notification** screen, choose your desired notifications.

To activate or deactivate Remember volume settings*

- On the **Hearing aid settings** screen, tap **Remember volume settings**, then toggle the switch.
By default, your hearing aids remember your last used volume settings for each program.

To activate or deactivate call handling*

- On the **Hearing aid settings** screen, tap **Answer calls faster**, then toggle the switch.
By default select hearing aids allow you to answer calls by double tapping your hearing aid.**

To activate or deactivate Keep connection***

- On the **Hearing aid settings** screen, tap **Keep connection**, then toggle the switch.
By default the toggle is on and you will connect faster when you reopen the app.

* Some app features are only available for select hearing aids

** To check for compatibility, visit: www.oticon.global/compatibility

*** Only available on Android devices

Find my hearing aids

If you are unable to find your hearing aids, the app can show you, on a map, the last known location of your paired hearing aids when the app was running.

To locate your missing hearing aids


1. On the tab bar, tap **Hearing aids**.
2. Tap **Find my hearing aids**.

The hearing aid icons indicate how close you are to each connected hearing aid.

On the map, the green marker indicates that there is connection between your hearing aid and device.

The red marker indicates that the connection is lost. Tap the icon to see the date, time and location of the last connection.

For best results, it is recommended to keep the app turned on, either actively being used, or running in the background.

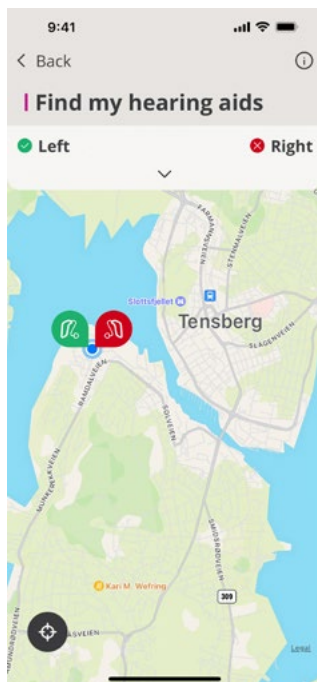

Apple Watch®

Introduction

With the Apple Watch app, you can control your hearing aids. From the Home screen, you can adjust volume, change programs and mute/unmute your hearing aids. You can also check your hearing aids' battery levels and toggle the Remote microphone OFF. In select hearing aids you can toggle the SpeechBooster ON/OFF.

Before you can use the app on your Apple Watch, you must first install the app on your iPhone and ensure you pair your hearing aids with your iPhone.

System requirements

To use the app with Apple Watch, you need the following:

- Apple Watch Series 4 or later
- WatchOS® 8 or later

Home screen

Your Apple Watch app opens on the home screen. Here you can adjust volume, change programs and mute/unmute your hearing aids.


To adjust the volume

- Above or below the volume indicators tap the +/- buttons to turn the volume up or down.
- Volume can also be adjusted using the Digital Crown if volume sliders are merged.

Be aware that in the volume level indicator, 0 is the startup volume prescribed by your hearing care professional.

Volume adjustments apply only to the selected program.

To mute/unmute your hearing aids

- On the app Home screen, between the volume indicators and below the program button, tap the  button to mute or unmute your hearing aids.

To change programs

1. On the app Home screen, between the volume indicators and above the mute button, tap the **P1** button.
2. On the Program screen, use your scroll to the desired program.
Alternatively, you can use the Digital Crown to change programs.
3. Tap the program to select it.


After you tap to select a program, the background color of the icon depicting the program, is filled in.

To check battery levels

- From the app Home screen, swipe to the right.

To activate/deactivate Remote microphone or SpeechBooster

1. From the app Home screen, swipe to the left.
2. Tap the toggle switch to activate/deactivate the desired function.
3. The toggle switch is GREEN when activated.

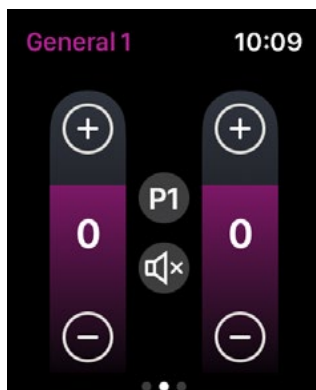

The Remote microphone can only be deactivated from your Apple Watch.

Apple Watch complications

To quickly access the app, you can set up the Apple Watch complication, if complications are available on the selected watch face.

Available complications can show your current hearing aid program and/or your hearing aid battery levels via an icon on the Apple Watch home screen.

Create account or sign in

To use some features of this app you will need to create an account or sign in with your existing user credentials.

If you already signed up to other Oticon registered services, you can reuse your credentials to sign in to your account.

You can create an account either by using your Apple ID, Facebook, or Google credentials or by using your email address.

To sign in with existing credentials

1. On your device, on the tab bar, tap **More**.
2. On the **More benefits** screen, tap **My Profile**.
3. Tap **SIGN IN**.
4. Follow the guidance in the app.
5. In the **You are signed in** screen, tap **GO TO THE APP**.
6. If using the **RemoteCare** feature, continue to *Prepare for your remote visit* section of this booklet.

To create an account

Using Apple ID, Facebook, or Google

1. On your device, on the tab bar, tap **More**.
2. On the **More benefits** screen, tap **My Profile**.
3. Tap **CREATE ACCOUNT**.
4. Follow the guidance in the app.

Create account or sign in (continued)

Using an email address

1. On your device, on the tab bar, tap **More**.
2. On the **More benefits** screen, tap **My Profile**.
3. Tap **CREATE ACCOUNT**.
4. In the email address field, enter your email address. Tap **CONTINUE**.
5. In your email inbox, the **Welcome to our app** email you received and click the **Verify your email** button. You are now directed to a webpage, where you must create a password. In the **Password** field, create a password. You will use this password when you sign in to the app.
6. In the **Confirm new password** field, reenter the password. Tap **CONTINUE**. You are now redirected to a page informing that your account is ready and that you must return to the app on your device and sign in.
7. Close your web browser, return to the app and then follow the guidance in the app.

Prepare for your remote visit

- Plug your device into power or ensure that your device is fully charged.
- Ensure your hearing aids are paired with your device.
- Ensure you enable Bluetooth on your device.
- Position yourself where there is a stable Internet connection.
- Insert new batteries in your hearing aids to ensure they are adequately powered. If you use rechargeable hearing aids, make sure they are fully charged. Ensure you have your charger nearby, as you may need it to restart your hearing aids.

Alternatively, you can manually restart your hearing aids by turning them **OFF** and **ON**.

Note

In order to maintain a strong and reliable connection for the duration of the remote visit, your hearing aids must have new or fully charged batteries.

- Turn OFF any type of battery-saving mode on your phone/tablet.
- Terminate other apps you are running to avoid disturbances.

During the visit, do not:

- Place your hearing aids more than 3 meters away from your device
- Turn OFF your hearing aids
- Turn OFF Bluetooth on your device
- Enable Flight mode on your hearing aids or your device
- Turn OFF your device
- Switch between Wi-Fi and data plan
- Terminate the app or switch to another app
- Enable any kind of battery-saving mode on your device

Begin visit

After you have gone over the **Prepare for your remote visit** checklist on the previous page, you must launch the RemoteCare feature in the app.

If you are not already signed in or if you do not have an account, follow the steps in the **Create account or sign in** section of this booklet before proceeding.

To begin your RemoteCare visit

1. On the tab bar, tap **More**
 2. In the **More benefits** screen, at the top, tap **RemoteCare**.
 3. In the **RemoteCare** screen, at the bottom, tap **GO TO VISIT***.
 4. In the **Connect your hearing aids** screen, tap **CONNECT**.
 5. In the following **Before the visit** screens, follow the guidance in the app and tap **CONTINUE** whenever prompted.
- If you have not paired your hearing aids to your device, you will be taken to the **Pair your hearing aids** screen.

If this happens:

- Click **PAIR** and follow the steps on the screen to pair your device to your hearing aids.

Waiting Room

- In the **Waiting Room** screen, wait for your hearing care professional to connect and join the visit. While you wait, you see a video stream of yourself.

* If this is your first time using the app, ensure you read and agree to the Data & Privacy notice. To activate the services in the app, tap AGREE.


Your RemoteCare visit

When your hearing care professional joins the remote visit, they appear on your screen in a video stream.

- A. **Camera:** Turn your own camera ON/OFF.
- B. **Audio:** Turn your own audio ON/OFF.
- C. **Chat:** Write a text message to your hearing care professional.

Hearing aids connected

When your hearing care professional establishes a remote connection to your hearing aids, the **L** and **R** graphics change from blinking to steady green.

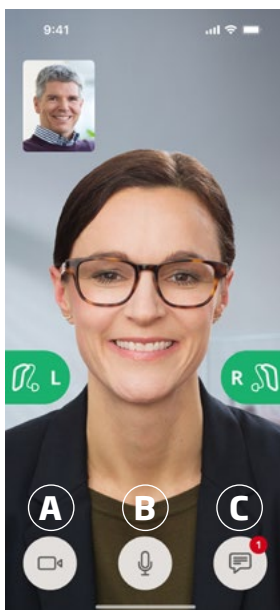

Hearing evaluation (In-situ audiometry)

This is an optional function of the RemoteCare feature which enables your hearing care professional to remotely check if your hearing loss has changed and to fine tune the settings of your hearing aids accordingly.

To ensure a successful hearing evaluation

- Ensure you are in a quiet place with little or no background noise. Too much background noise can affect the results, possibly requiring the hearing evaluation to be redone.
- Wear your hearing aids correctly and as advised during your first fitting.
- Do not turn off or unpair your hearing aids during the visit.

IMPORTANT NOTICE

If your Internet connection is lost during the hearing evaluation visit, you will receive an on screen notification and the app will attempt to reestablish the connection.

If you hear a tone in your hearing aids after the Internet connection is lost

- Remove and restart your hearing aids if they are muted or still playing sound.

In some cases, during a RemoteCare visit, the app goes into recovery mode and updates your hearing aids with the latest settings from your hearing care professional.

Applying settings

When your hearing care professional uploads new settings or configurations to your hearing aids, the **L** and **R** graphics change color and blink orange.

Settings saved

When the new settings have been successfully saved to your hearing aids, the **L** and **R** graphics change color to steady green again.

Restarting hearing aids

When you and your hearing care professional agree to end the visit, then the hearing care professional restarts your hearing aids (by turning them **OFF** and **ON**). If enabled, your hearing aids play a melody or jingle.

End visit

When your hearing care professional ends the visit, and the **Visit ended** screen appears, tap **OK**. At this point, you can begin using your hearing aids.

Intended use of Tinnitus SoundSupport

Tinnitus SoundSupport is a tool intended to generate sounds to provide temporary relief for patients suffering from tinnitus as part of a tinnitus management program.

The target population is adults (over 18 years old).

Tinnitus SoundSupport is targeted to licensed hearing care professionals (audiologists, hearing aid specialists, or otolaryngologists) who are familiar with the evaluation and treatment of tinnitus and hearing loss. Tinnitus SoundSupport must be fit by a hearing care professional participating in a tinnitus management program.

Tinnitus SoundSupport is a sound generator that can be integrated into your hearing aids. Specially designed programs in the hearing aids allow you to play various sounds that can help you move your attention away from your tinnitus.

During the fitting session, your hearing care professional can enable the Tinnitus SoundSupport in one or more programs. They can be accessed along with the other programs on top of the screen.

For information about limitation of use of these programs, ensure you read your hearing aid's Instructions for use, or contact your hearing care professional.

Note

Tinnitus SoundSupport feature is only available in the app if Tinnitus has been activated by the hearing care professional in your hearing aids.

Tinnitus SoundSupport


The Comfort pulse feature lets you control the sound for Tinnitus SoundSupport. You can choose a constant tone or one of four different sound pulsations.

To select a tinnitus program with Comfort pulse sounds

1. On the tab bar, tap **Sound**.
2. On the navigation bar, in the program selector, tap the down arrow, then tap the tinnitus program you would like to use.

For more information about programs, see the **Change program** section.

To adjust tinnitus Comfort pulse sound level

1. On the navigation bar, tap the **Sounds** tab.
2. Above the tab bar, within the row of buttons, tap the Comfort pulse button .
3. On the **Choose a sound pattern** screen, tap to choose one of the Comfort pulse options.
4. Move the slider(s) up or down.

You can do this for each hearing aid independently or both hearing aids at the same time.

For more information about volume control, see the **Change volume** section.

To adjust the volume of ambient environmental sounds picked up by your hearing aids

1. On the navigation bar, tap the **Surroundings** tab.
2. Move the slider(s) up or down.

General warnings

For your personal safety and to ensure correct usage, you should familiarize yourself fully with the following general warnings before using your device. Consult your hearing care professional if you experience unexpected operations or serious incidents with your device during use or because of its use. Your hearing care professional will support you with issue handling and, if relevant, reporting to the manufacturer and/or the national authorities.

Using the app can distract you in situations where full concentration is required. Be aware of your surroundings while using the app.

Troubleshooting

This Instruction for use document contains detailed guidance on the use of the app. If you experience unexpected behavior when using the app or have additional questions not covered by this document, you should consult the Troubleshooting section within the app.

To access the Troubleshooting section

1. On the tab bar, tap **More**
2. In the **More benefits** screen, near the middle, tap **Help & Support**.
3. In the **Help & Support** screen, at the top, tap **Troubleshooting**.

Troubleshooting RemoteCare

I can only see myself, there is no video stream of my hearing care professional.

1. Let your hearing care professional know that you cannot see them. Your hearing care professional will attempt to refresh the video connection.
2. Wait for a short while, usually the video stream is re-established automatically.

What do I do if I receive a phone call during the RemoteCare visit?

It is recommended that you decline phone calls during the RemoteCare visit.

What do I do if my Internet connection suddenly stops working?

Attempting to reconnect happens automatically. If the reconnection fails, you will have the option to **END THE VISIT**.

During a visit, I left the app and returned to it. Does that have any impact?

While your app is not active, your hearing care professional cannot see you, and it may seem to them as if you have a connection issue.

I am having challenges with the visit with my hearing care professional, e.g. video is unstable.

This is most likely due to an unstable Internet connection. Try to repeat the visit with a better Internet connection.

The visit suddenly ended without a mutual agreement with my hearing care professional.

Try to reconnect to the visit or contact your hearing care professional.




The settings in my hearing aids cannot be recovered:




1. Ensure Bluetooth is enabled on your device.
2. Ensure you insert new batteries or that your rechargeable hearing aids are fully charged.
3. Ensure your hearing aids are paired with your phone or tablet.
4. Restart the app.

The app checks if your hearing aids are in working order. In some cases, the app goes into recovery mode and updates your hearing aids with the latest settings from your hearing care professional. Please follow the guidance in the app. If you still cannot recover the settings in your hearing aids, ensure you contact your hearing care professional.

Technical information

The following are definitions that may appear in the Instructions for use and on the **About the app** screen for Oticon Companion:


Description of symbols used in this booklet	
	Warnings Text marked with a warning symbol must be read before using the app.
	Manufacturer The device is produced by the manufacturer whose name and address are stated next to the symbol. Indicates the medical device manufacturer, as defined in EU Regulation 2017/745.
	CE mark The device complies with all required EU regulations and directives. The four digit number indicates the identification of the notified body.

Description of symbols and abbreviations used in the app	
	Consult Instructions for use Indicates the need for the user to consult an Instructions for use. The symbol may be accompanied by an indication on where to locate the instructions for use.
	Medical Device The device is a medical device.
	Global Trade Item Number A globally unique 14-digit number used to identify medical device products including medical device software.

This medical device complies with Medical Device Regulation (EU) 2017/745.

Declaration of Conformity is available at headquarters:

Oticon A/S
Kongebakken 9
DK-2765 Smørum
Denmark
www.oticon.global

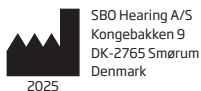

Apple, the Apple logo, iPhone, iPad, Apple Watch, Mac and the Mac logo are trademark of Apple Inc. registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android™, Google Play and the Google Play logo are trademarks of Google LLC.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc..

Any use of such marks by Demant is under license. Other trademarks and trade names are those of their respective owners.

